## Scan for more information

## Check out our learning platform Reliance U! www.RelianceU.tech

## Reliance Orthodontic Products, Inc.

1540 W. Thorndale Avenue, Itasca, IL 60143 USA
1-800-323-4348 | Made in USA | Rx Only
RelianceOrthodontics.com

**EMERGO EUROPE**  
Prinssessgracht 20, 2514 AP The Hague, The Netherlands

**AUSTRALIAN SPONSOR EMERGO AUSTRALIA**  
Level 20, Tower II, Darling Park, 201 Sussex St, Sydney, NSW 2000, Australia

## Reliance Orthodontic Products, Inc.

## ORTHO FLEXTECH®

## INSTRUCTIONS FOR USE

RelianceOrthodontics.com

### EN - ENGLISH

**INDICATIONS FOR USE:** Intended for use as a Lingual Retainer Wire. This product is not indicated for individuals with sensitivity to Nickel.

**INSTRUCTIONS FOR USE:**

- Prophy and sandblast the enamel, rinse and dry. Etch the enamel for 30 seconds. Rinse and Dry.
- Apply one coat of Assure® Plus to the etched enamel. Lightly air dry.
- Apply a minimal amount of L.C.R.™ GoTo™ or FlowTain™ to the lingual surface of each tooth.

- Place the Ortho FlexTech® chain in the applied adhesive (“undercoat”) on the lingual surface. Position for best alignment and take up any slack in the chain.

- Alternate Method: Place one end of Ortho FlexTech®chain in the applied adhesive paste (“undercoat”) on the lingual cuspid surface. Position for best alignment and light cure to hold chain in place. Proceed to the next tooth, taking up any slack in the chain. Repeat this process until the opposing cuspid has been reached.

- Place an “overcoat” of L.C.R.™ GoTo™ or FlowTain™ paste on top of the cured material to cover the Ortho FlexTech™ chain. Light cure the “overcoat” paste on each bonded tooth.

- Check for interference between upper and lower. Polish any rough spots or high points carefully.

**NOTE:** If used on upper to hold diastema, consider secondary .0175 twist 1 to 1 or Retanium™, as the chain can stretch slightly allowing space to reopen.

Ⓜ Do not Re-Use. This is a single use device
**REORDER NUMBERS:** #OFT, #OFT2, #SSOFT, #SSOFT2

**WARRANTY:** Reliance Orthodontics Products, Inc. recognizes its responsibility to replace products if proven to be defective. Reliance Orthodontic Products, Inc. does not accept liability for any damages or loss, direct or consequential, stemming from the use of or inability to use the products as described. Before using, it is the responsibility of the user to determine the suitability of the product for its intended use. The user assumes all risk and liability in connection therewith.

**RX ONLY:** U.S. Federal law restricts this device to sale by or on the order of a dental professional.

**NOTICE:** Report any serious incident to Reliance and the competent authority of the state in which the user and/or patient is established.

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